


## Face Mask Instructions (Project COVID-19)

# Materials

### Required Materials:

- 4 Pieces of String, 15" Long (Or similar material. 2 pieces 7" if using rope elastic)
- Surgical wrap material or tight-knit cotton 12" x 9"
- Ruler
- Pencil
- Scissors
- Iron

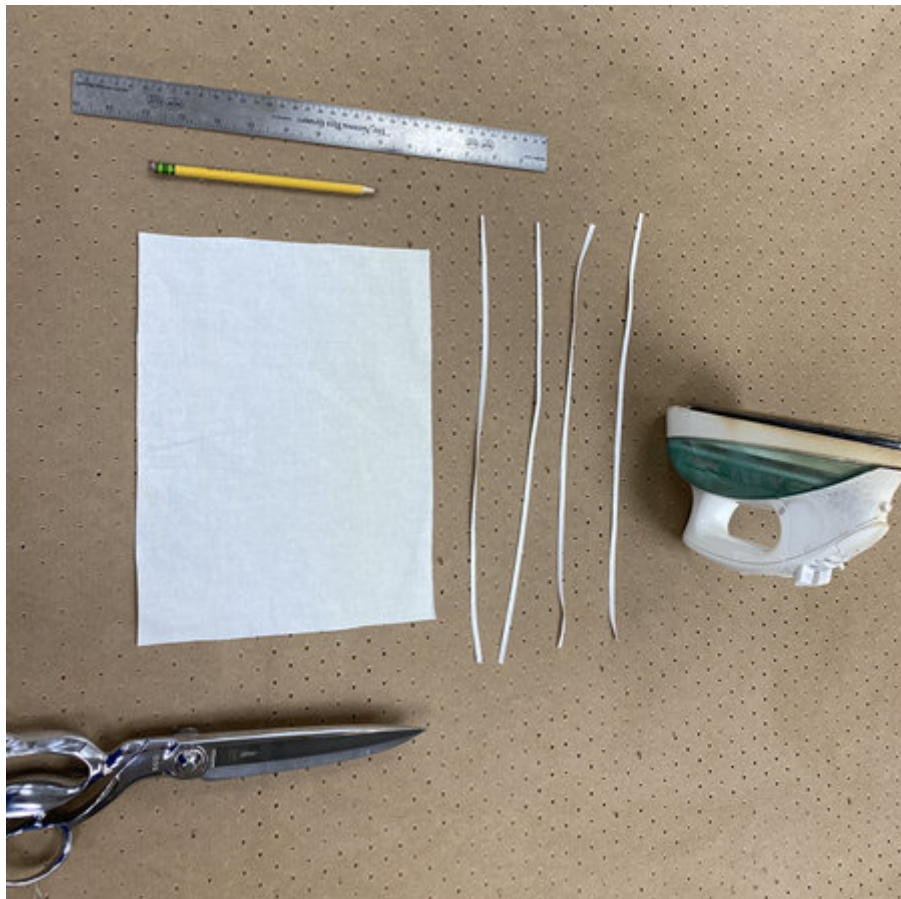


FOR FURTHER INFORMATION  
PLEASE EMAIL [✉ alex@adrianjules.com](mailto:alex@adrianjules.com)

## Face Mask Instructions (Project COVID-19)

# Instructions

### Step 1:

Take the 12" edge of the material and fold back ½". Use the iron to create a crease. Do this for both sides.

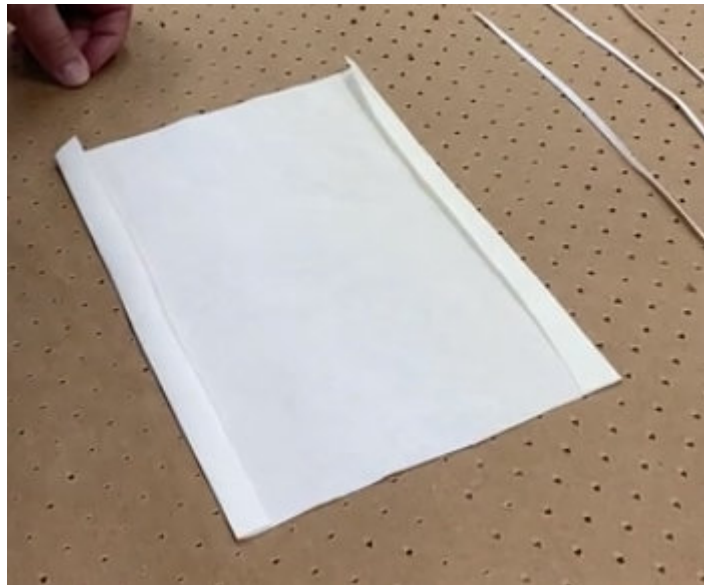

### Step 2:

Fold the material in half so that both of the unfolded ends (9" sides) are touching.



## Face Mask Instructions (Project COVID-19)



### Step 3:

Sew down the open side (9" Sides touching). Make sure to back tack at the beginning corner and end corner.



### Step 4:

Turn the material inside out.



## Face Mask Instructions (Project COVID-19)


### Step 5:

Put one string in each of the four corners. Make sure to go over this a few times (tack 3 times) because the corners will be a point of stress.



### Step 6:


Marking of the pleats. First mark 1 ½" from the top. Next three marks will be 1 inch down from each other.

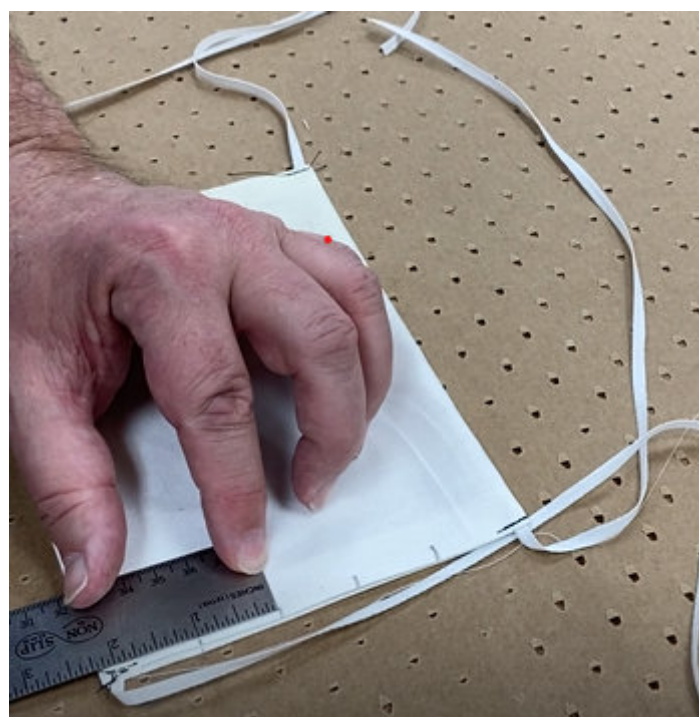

## Face Mask Instructions (Project COVID-19)


### Step 7:

Grab the top of the mask, and fold so that the first mark lines up with the second mark, creating a pleat (fold). Use your iron to crease that and hold that in place.

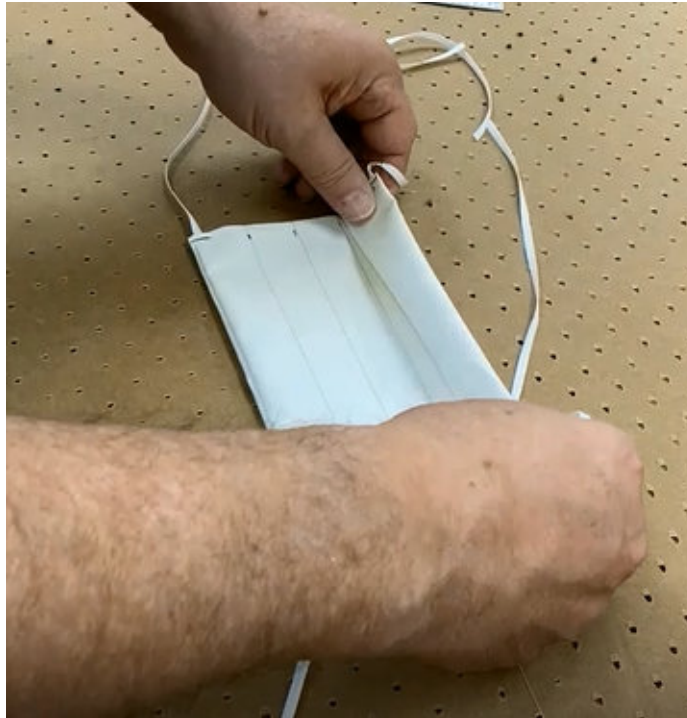

### Step 8:

Now, take the third mark and match it to the fourth mark. Use your iron to crease this fold and hold it in place.

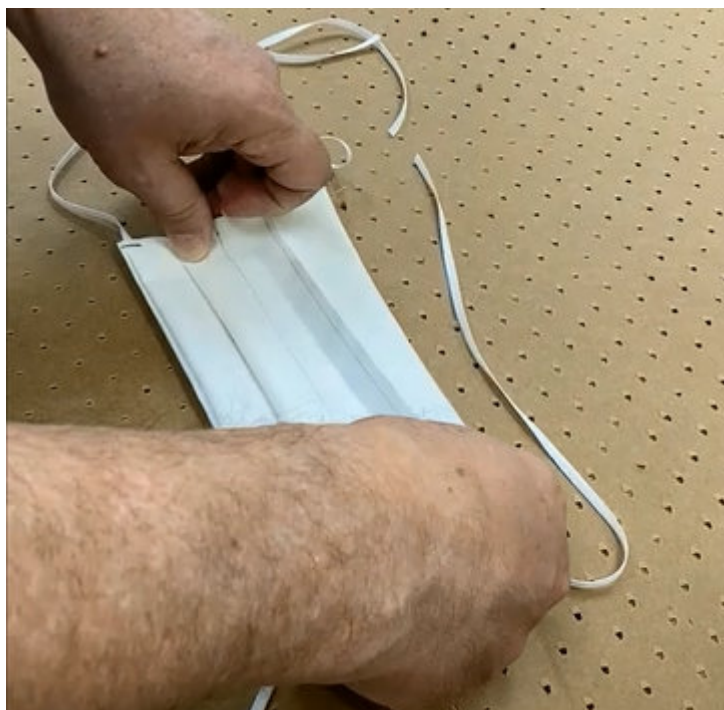

## Face Mask Instructions (Project COVID-19)

### Step 9:

Repress the folds/creases if needed to keep them in place.



### Step 10:

Sew the two side seams of the mask where the folds are created. It is easiest to have the folds pointed down when sewing. Make sure to tack at the beginning and end.



# Face Mask Instructions (Project COVID-19)

## Mask Completion

